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User manual
Rollz cane holder

Rollz International BV – Rollz Motion

© Copyright 2011 Rollz ® International BV
Information contained in this document may be changed without prior notice. Rollz ® International BV cannot be held responsible for technical faults or defects.

The purchased product may differ from the product described in this user guide. For the most up-to-date user guide, contact Rollz International BV or go to www.rollz.com

Contents

Introduction	18
Important information	18
Warning	19
Check	19
Usage	20
What goes where?	20
Attaching your Rollz cane holder your cane	21
Attaching your Rollz cane holder to the frame	23
Removing your Rollz cane holder	25
Cleaning	25
Comments	25
Guarantee	26

Introduction

The Rollz cane holder

You have opted for the Rollz cane holder to go with your Rollz Motion. Keep on going without effort. Even now. That's what Rollz is all about. Which is why Rollz designed the stylish Rollz Motion to take you wherever you want, for as long as you want. And the Rollz cane holder allows you to take your cane or crutch along for the ride.

This cane holder is meant for the Rollz Motion only.

Important information

- The Rollz cane holder is designed for canes and crutches with diameters from 19 - 25 mm.
- The Rollz cane holder is designed for use with the Rollz Motion only. Using this product on other rollators can lead to a damaged cane holder or rollator and to physical injuries.

WARNING!

- Before use, read the manual carefully and put it in a safe place for future use.
- Only use canes and crutches the cane holder is designed for (see: important information).
- Always remove canes and crutches before folding or lifting the Rollz Motion, to prevent your cane or crutch from falling.

Check

Check the contents of the package and compare it with the list below. Should anything be missing, you can contact Rollz International (see: Contact Information).

The Rollz cane holder package contains:

1 cane clamp

1 frame clamp

3 rubber bands (small, medium, large)

1 manual

Use:

What goes where?

Fig. 1

1. Cane clamp
2. Frame clamp
3. Rubber band
4. Bevelled side
5. Gap
6. V-shape

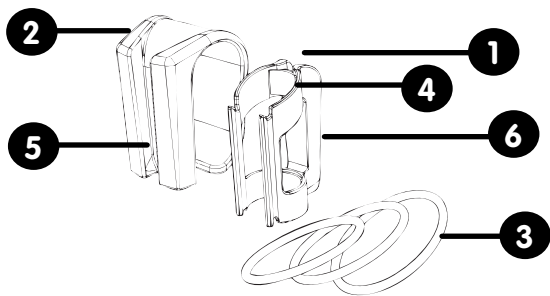
figure 1

20

EN



Attaching your Rollz cane holder to your cane

The Rollz cane holder comes with three rubber bands, one small, one medium and one large (Fig. 1). Depending on your cane's or crutch's diameter you can use one of these rubber bands to fasten the cane clamp.

1. Hold the cane clamp with the opening towards your cane or crutch, with the bevelled side upwards. Fasten the cane clamp by pushing it around your cane or crutch.
2. Slide the cane clamp to the proper height, where it will not get in contact with the front wheels. Which is at approximately 52 cm from the foot of your cane or crutch (Fig. 2).
3. Now use one of the rubber bands to fasten the cane clamp around your cane or crutch.

ATTENTION: Always check that the cane clamp is properly fastened. This means the cane or crutch is unable to slide once you have fastened the rubber band.

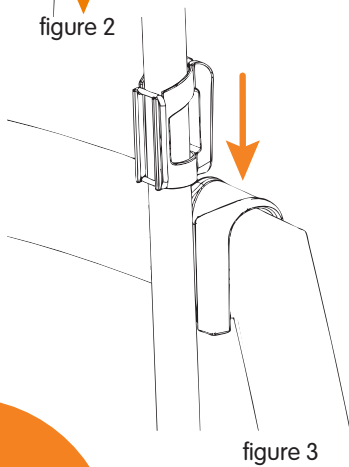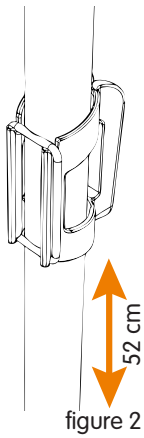22
EN

Attaching your Rollz cane holder to the frame

1. Hold the frame clamp with the flat side on the inner side of the Rollz Motion above the frame.
2. Click the frame clamp on the frame of the Rollz Motion by pushing the frame clamp straight down onto the frame. Always make sure that you put the frame clamp on the front side of the Rollz Motion (Fig. 3).
3. Take your cane or crutch with the attached cane clamp and hold it with the V-shape towards the Rollz Motion. Now slide it downwards into the gap of the frame clamp (Fig. 3).
4. Check that your cane or crutch is firmly attached, does not block the wheels and does not slide downwards (Fig. 4).

PLEASE NOTE: Always check that the cane holder is fully intact. Contact your supplier if the cane holder is broken or damaged.

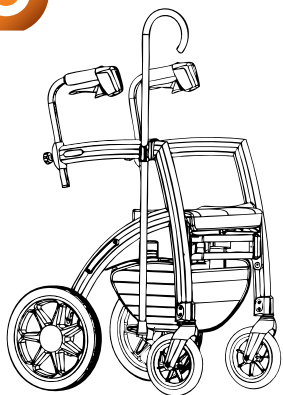
figure 4

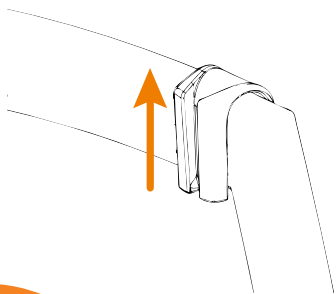
figure 5

Removing your Rollz cane holder

1. Slide your cane or crutch plus cane clamp out of the frame clamp.
2. Firmly grip the frame clamp on both sides at the bottom.
3. Push from the bottom the frame clamp from the frame (Fig. 5).

Cleaning

You can clean your Rollz cane holder with a cloth and lukewarm water.

Comments

Rollz International continually endeavours to improve its products. We therefore always highly value any feedback with regard to design, user-friendliness, materials, instructions etc.

If you have any feedback for us, you can contact us at:

Rollz International BV
Pedro de Medinalaan 63
1086 XP Amsterdam
The Netherlands
+31 (0) 20 362 20 10
info@rollz.com

Guarantee

Rollz International BV, located in Amsterdam (hereinafter referred to as "Rollz"), in its capacity as manufacturer, provides a guarantee on the Rollz cane holder (hereinafter referred to as "the product") in accordance with the following conditions:

1. The guarantee period is 24 months following the date of purchase. The guarantee period is never extended, even following repairs.
2. The guarantee entitles the purchaser to repair work, but does not entitle the purchaser to exchange the product or return the product for a refund.
3. In cases in which the repair is covered by the guarantee, Rollz is responsible for the risks and costs involved in transferring the product from and to the store from which you bought it (hereinafter referred to as "the supplier").
4. The guarantee can only be invoked if you provide the faulty product to Rollz, together with the following:
 - a copy of the dated purchase receipt
 - a clear description of the fault(s)
5. The guarantee is non-transferable and can only be invoked by the initial purchaser.
6. The guarantee does not detract from your legal rights, e.g. the right to make a claim against your supplier with regard to any possible faults.

The guarantee does not apply to the following situations:

1. If you have performed repairs or have instructed anybody to perform repairs upon the product without the prior written permission of Rollz or the supplier.
2. If faults are caused by user negligence, accidents and/or usage, or maintenance other than the instructions contained in the instruction manual.
3. If the serial number is damaged or has been removed.
4. If the product has been damaged as a result of standard wear and tear.

Important note:

The guarantee period runs for 24 months following the date of purchase. In the event that a fault is discovered, you must inform your supplier.

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Nehmen Sie über die nachstehende Adresse Kontakt mit Rollz Deutschland GmbH auf:
Rollz Deutschland GmbH